




# Welcome To Your **Best Self!**

The world's first non-invasive device to improve erectile function used in the comfort of your home.

Now available with a cloud-based mobile app for real-time feedback and personalised treatments

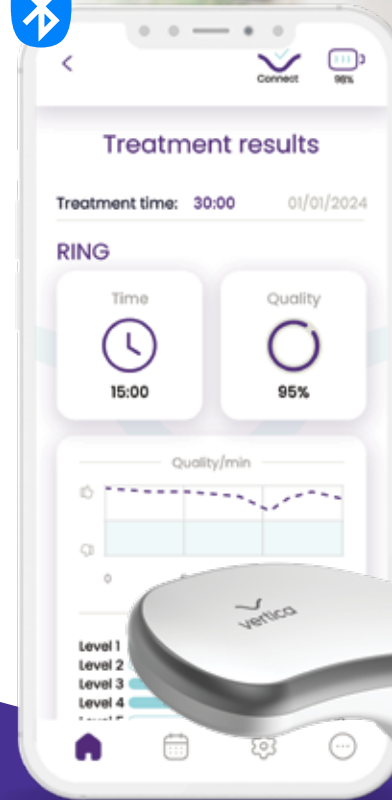

# Join the Thousands of Vertica® Users

While many men would rather not discuss the subject, research shows that from the age of 40, many men begin to experience reduced erectile function.

We have developed Vertica® together with experts in urology and advanced technologies, as a non-invasive, home use device to improve erectile function and treat erectile dysfunction (ED). Thousands of men are already benefiting from their treatments with Vertica, improving their quality of life.

We've put together some information to help you decide if Vertica is the right treatment for you. If you are looking for more information, **visit our web site or drop us a mail.**



# Vertica is a non-invasive medical device used in the comfort of your home

Vertica improves erectile function without the need for drugs, with no side effects and no prescriptions. Vertica is used at home at your convenience and has proven long-lasting results. Vertica enables spontaneous relations, enabling you to be **'always ready'**.



# Vertica leads in all parameters

Vertica treats the root cause of reduced erectile function. The majority of the available solutions, such as chemical and natural drugs, vacuum pumps, rubber rings, shock-wave devices, penis injections, and penile implants treat the symptom of erectile dysfunction – they do not even attempt to treat the root cause.

Vertica is different. It targets the physiological mechanism that controls the erection and regenerates the damaged tissue.

There are solutions that aim to increase blood flow into the penis to create erections (in-flow), but they don't stop blood from flowing out (out-flow). Vertica addresses both issues at the root cause and our users report they are already experiencing improved erections, that last longer.




	Home Use			In-Clinic	
	Vertica	Medication	Vacuum Pump	Injections	Shockwave (ESWT)
Treats inflow and outflow	✓	✗	✗	✗	✗
Non-invasive	✓	✗	✓	✗	✓
Spontaneity	✓	✗	✗	✗	✓
No prescription required	✓	✗	✓	✗	✓
Discrete, private use	✓	✗	✗	✗	✗
Maintains erectile function long term	✓	✗	✗	✗	✗
Rejuvenates collagen in the tunica albuginea and in the corpora	✓	✗	✗	✗	✗

# Improve your Erectile Health with Vertica

During the Vertica treatment you will feel a gentle warm sensation in your penis and perineum area (between your anus and scrotum). Our users report experiencing a pleasant warming sensation when Vertica is used.

**Since Vertica is used at home, men maintain optimal function by using Vertica regularly, similar to going to the gym to keep fit. Much like maintaining good body tone requires ongoing exercise, we suggest you continue your Vertica regimen to maintain good erectile functioning. Once it becomes part of your workout, you will notice the difference. That's because Vertica actually rejuvenates the tissues involved in maintaining erections.**

Usually, men start to notice an improvement within the first two to three weeks of treatment; some even experience 'morning wood' and spontaneous erections throughout the day. The complete initial round of treatment usually lasts four weeks to 3 months.

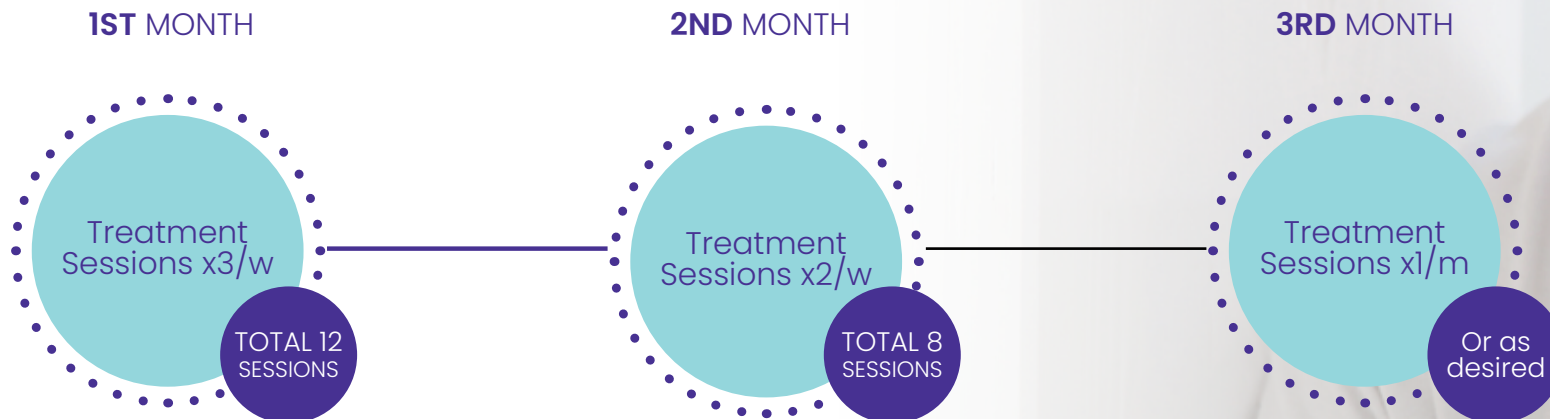


# Vertica Now with the Advanced **Verticaplus App**

## Breakthrough Treatment for Erectile Dysfunction

A clinical study published in the IJR journal assessed the safety and efficacy of Vertica and found that all participants achieved positive results.




Erectile Hardness Scores (EHS 3 + EHS4)

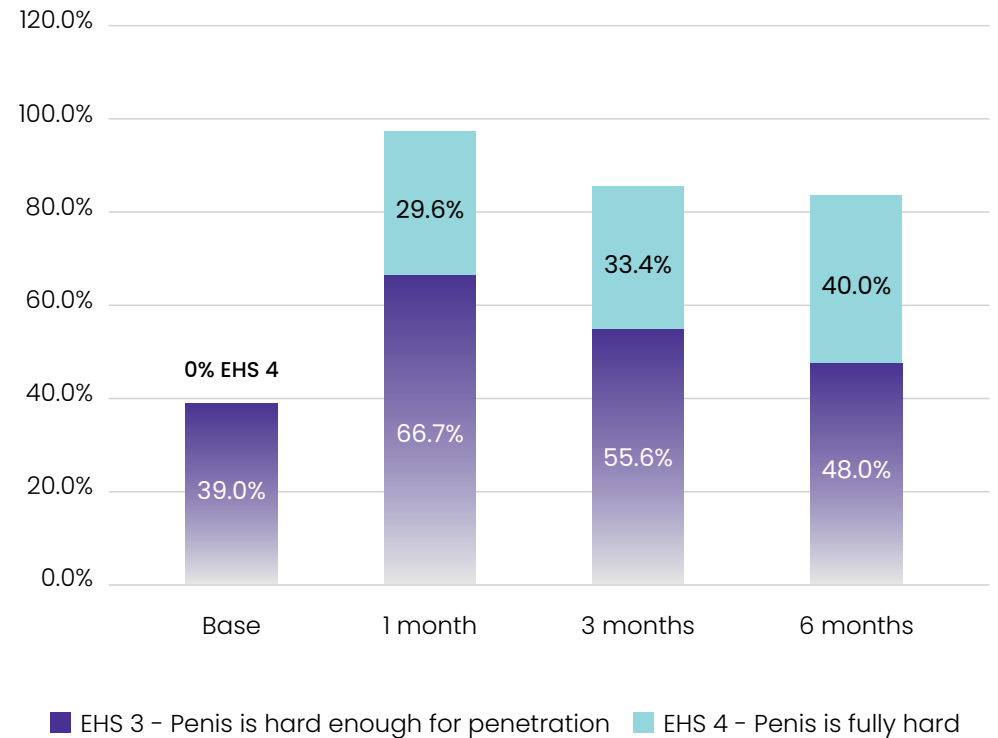

# Reach Optimal Results with the **Verticaplus App**

The new VerticaPlus supercharges the benefits of the Vertica with the addition of the VerticaPlus App which adds enhanced user experience with ongoing treatment feedback, guided tutorials, and personalised parameters.

**VerticaPlus** is the next-generation medical device that treats erectile dysfunction without the need for drugs, no side effects, and has long-lasting results. VerticaPlus is used at home 2-3 times/week for the first 2 months and then, as necessary, for maintenance.

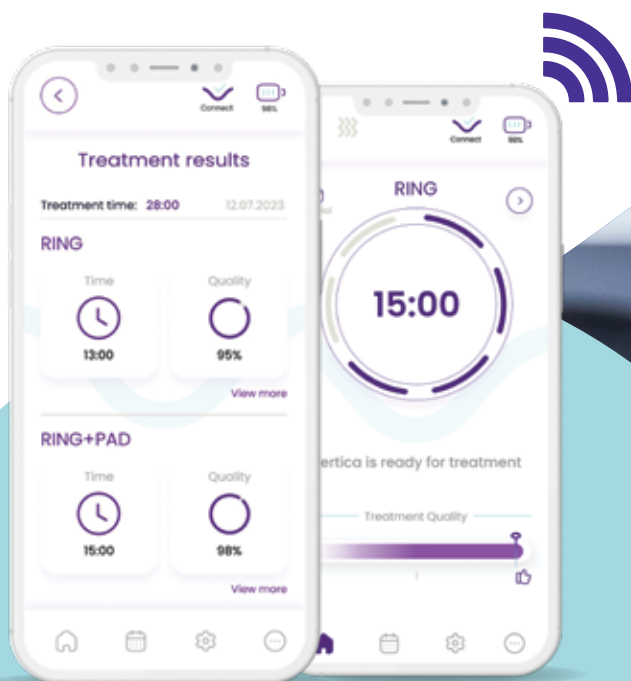

# Gain Confidence with **Realtime Feedback** and Attain **Optimal Results** with a Range of **Productivity Tools**

The cloud-based app provides Vertica Customer Care Team with the necessary data to provide actual, real-time guidance and support, to give you the confidence that you are getting the most effective treatments.

The VerticaPlus App is your companion during the treatment, counting the effective treatment time, and quality of the treatment.

You can check your monthly schedule using the calendar, and review 'How to Treat' procedure using the online tutorials at every stage of the treatment.



# In the Words of Our Customers

I've been using the Vertica device now for two months as recommended. During the first month, I used it 3 times a week, and in the second month, I used it twice a week.

I want to highlight that I am experiencing significant improvement, with longer erections, and I can now have relations twice a week. In the morning when I wake up I have a morning erection like I did when I was 18.

Another thing is that my libido returned, which had been dormant for almost a decade. I have diabetes, I don't drink, and I am afraid to take any pills – so for me this is a genius invention and has given me a reason to live.

Thank you!

**H, 66 years old**



I am 69 years old and wanted to boost my performance and enjoy life to the fullest. I got a Vertica device 10 months ago. I followed the recommended treatment plan and within two months, I saw amazing results - my erections became stronger and more satisfying. Now I don't worry or stress about anything when it comes to intimacy. I just let nature take its course... and best of all, I don't need any pills or chemicals to make it happen.

Feeling confident is a game-changer. It makes me feel more alive and active. And that's what Vertica does for me.

If you want to experience the same, don't hesitate to try the device. You won't regret it.

**G, 69 years old**





I really have to say that Vertica is really making a difference. I was a competitive athlete before I developed severe erection problems. You can't imagine what erection problems can do. I was a happy person who loved life. After the erection problems, I was a wreck. I can say 100% that Vertica does help and I thank you from the bottom of my heart for the Vertica device.

**M, 33 years old**

# Dedicated Customer Care

Our devoted Customer Care team of local operators responds to all inquiries and answers each question with professional and supportive answers. We are available for all your questions to offer guidance and support.

Reach out to us any way you feel comfortable.

 +44 204 525 1389

 [service\\_uk@vertica-labs.com](mailto:service_uk@vertica-labs.com)

 [vertica-labs.com](https://vertica-labs.com)

